PESAPERSONE MECCANICA A STRADERA MOD C201/202
MECHANICAL WEIGHING SCALE WITH STEELYARD MOD. C201/202
BÁSCULA PESA PERSONAS MECÁNICA ROMANA MOD. C201/202

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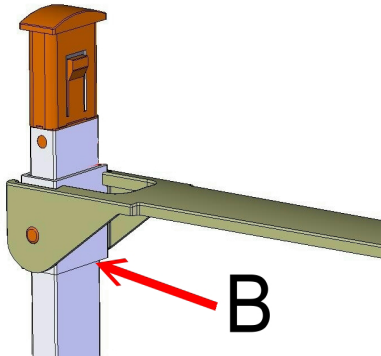
By choosing the electronic professional column weighing scale **WUNDER** mod. **C201** and **C202** you have bought a high precision tool.

For more than 40 years, **WUNDER**'s experience has been at the service of health. The mechanical weighing scale Wunder mod. **C201** and **C202** complies with the standards in force in hospitals, clinics and treatment centres.

The weighing scales **C201** and **C202** are class Im medical products with measurement device and are balanced in compliance with precision class III (according to standards 93/42 and 90/384 EEC).

Contents

1. GENERAL SPECIFICATIONS	16
2. SAFETY	17
3. DESCRIPTION AND EXPECTED USAGE	17
4. TECHNICAL SPECIFICATIONS	17
5. INSTALLATION	18
6. USAGE INSTRUCTIONS	20
7. MAINTENANCE AND SUPPORT	21
8. COMPLIANCE	21
9. NAMEPLATES	21
10. DISCARDING AND DISMISSAL	22
11. WARRANTY	23
12. ASSEMBLY AND USAGE OF THE MECHANICAL TELESCOPIC STADIOMETER (OPTIONAL)	24

1. GENERAL SPECIFICATIONS

 **CAUTION! Carefully read the manual before using the tool, since it provides important information on USAGE AND MAINTENANCE SAFETY.**

The descriptions and illustrations contained in this manual are not binding.

WUNDER reserves the right to make any change required for improvement with no need to update the documentation.

The Customer Service is available for any further information on product usage, maintenance and assistance.

Symbols:

The following symbols have been used in the manual:

MEDICAL DEVICE AND ELECTROMAGNETIC COMPATIBILITY

MEASUREMENT DEVICE

PRECISION CLASS

WARRANTY CERTIFICATE

CAUTION! PLACED BEFORE THE PROCEDURES ARE OVER. NON-COMPLIANCE CAN CAUSE DAMAGES TO THE PRODUCT

WARNING! PLACED BEFORE THE PROCEDURES ARE OVER. NON-COMPLIANCE CAN CAUSE DAMAGES TO THE PRODUCT AND/OR INHURIES TO THE OPERATOR OR PATIENT

WEEE WASTE

2. SAFETY

The operators must read this manual carefully and follow the instructions contained in it. They must also learn the correct tool usage and maintenance procedures.

The manufacturer is not responsible for any direct or indirect damage, included loss of profit, or for any other business-related damage which is caused by a usage due to the fact that the tool's usage is non-compliant with the prescriptions of this manual.

- Keep this manual for future reference and as a support for staff training.
- Do not overload the device beyond its maximum capacity value.
- Do not apply loads abruptly.
- Do not press the keys using sharp or cutting objects.
- Do not try and open the tool.
- Do not remove the seals on the device.
- Do not immerse the device in water or in other liquids.
- Regularly carry out maintenance operations and metric checks.

3. DESCRIPTION AND EXPECTED USAGE

The tool has been designed to weigh people in a clinic for general diagnostic purposes.

Usage environment: hospitals and medical clinics. The installation room must be equipped with an electric system in compliance with the legislation in force. Always use the weighing device in a place which is not exposed to magnetic interferences.

People authorised to use the product: operators and doctors who are familiar with the product.

Supervising and responsibility: the tool must be used only under the supervision of a qualified doctor or qualified personnel in charge of maintenance and periodical checks.

Limitations of use: the medical device can only be used as described in the manual.

4. TECHNICAL SPECIFICATIONS

Model	C201 column	C 202 column
Capacity	100-200g/ 450lb	220 kg/ 485lb
Category	50-100g	100 g
Manufacturer	Wunder Sa.bi.srl Trezzo S/Adda Milano Italy	
OIML approval	Class III	
Measurement unit	Kg/ lb	
Operation temperature	0°C - 35°C	
Accessories included	Telescopic stadiometer 60-210cm / 24-84 inches 5mm/ 0.25 inches	

5. INSTALLATION

The packaging kit contains:

1. Column and head with weighing system
2. Column anchor rod (rod inside the column)
3. Base with rubber platform
4. Telescopic stadiometer measurement height
5. Wheels for transportation
6. Mounting kit 8 various screws and 1 key
7. Instruction manual


STAGE 1: Assembling the column base

 **CAUTION!** Keep the base and the platform close to each other during the tipping, in order to prevent the levers bushing to stick out.

Place the scales column in horizontal position on a table (Fig. 3)

1. Untie the cables which block the levers.
2. Take out the anchor rod using the key and hook the base long lever until it is linked to the column anchor rod (Fig. 3-1)

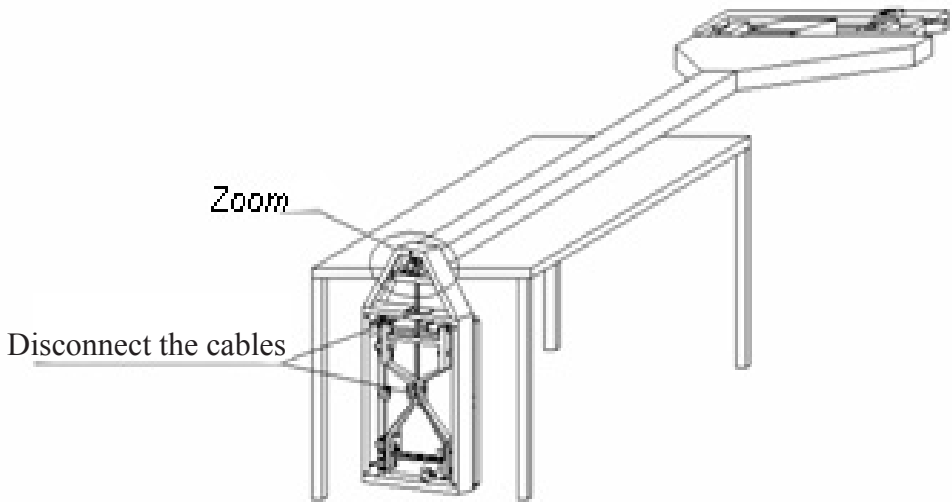Carefully place the scales on the ground.




Connection between steelyard and long lever

Fig 3


STAGE 2: Connecting the column anchor rod and the base lever

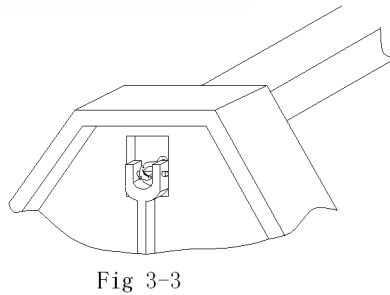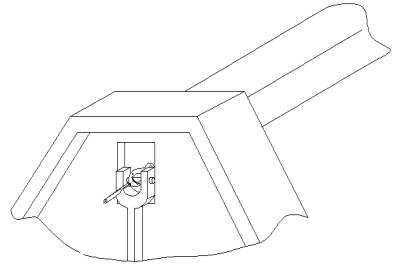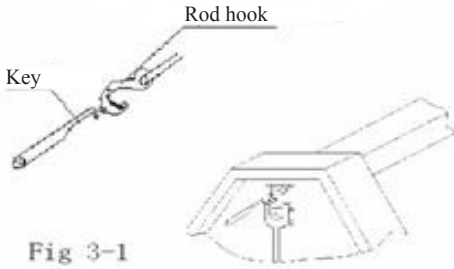Connect the column anchor rod to the long lever using the hooked key provided in the package (FIG. 3-1/ 3-2/ 3-3)


STAGE 3: Fixing the column / base (platform)

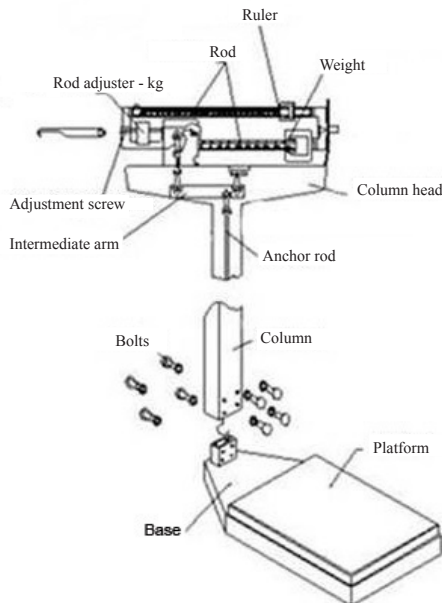Place the weighing scale on the ground and fix the column to the base using the four screws provided.

6. USAGE INSTRUCTIONS

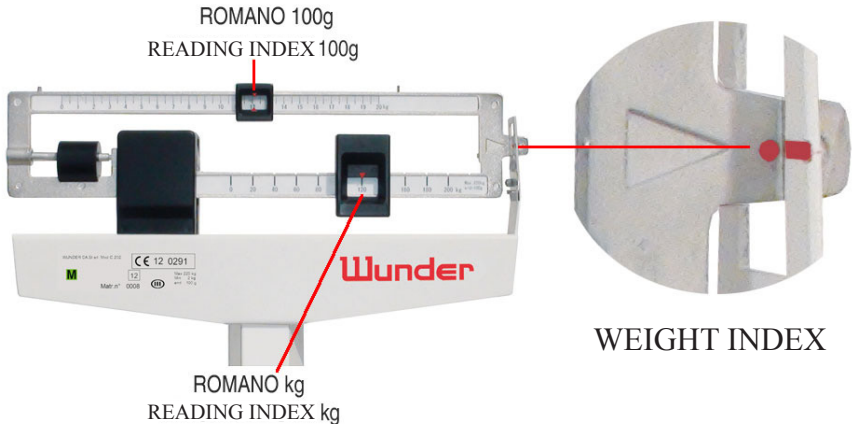1. Place the tool on a flat surface to ensure weight measurement precision
2. Check if the weight reading index is set to zero.
3. Go on the base (platform) of the scales.
4. Place the "Weight kg" for kg measurement and the "Weight 100g" for 100g measurement.
5. When the balance rod remains suspended and the index is on the red sign, the weight is correct.
6. Read the weight using the two weights: kg and 100g.
7. Step slowly out of the platform and reset the two Weights to zero.

7. MAINTENANCE AND SUPPORT

 **WARNING!** Regular maintenance interventions are recommended. The periodicity should be defined by the user.

Maintenance operations must be carried out by qualified personnel.

For further details, contact the **WUNDER** customer support centre.

For product better and longer life, a regular and accurate general cleaning is recommended.

The tool should be cleaned using a soft cloth, dampened with water or neutral cleanser. Avoid the usage of solvents or abrasive products.

In case the tool is not used for a long time, remove the batteries in the terminal and cover the battery to avoid damaging it.

During transportation, ensure the tool is not subject to impacts or high mechanical stresses.

For repairs and support, contact your reseller.

8. COMPLIANCE

WUNDER WEIGHING MACHINE: MODEL C201/C202

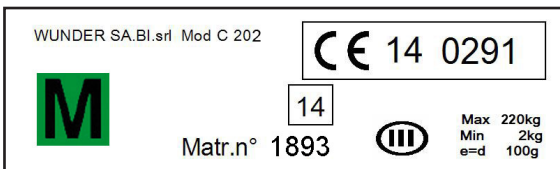
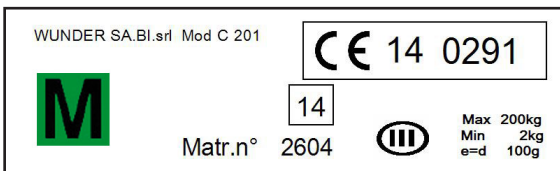
We certify that the tool has been checked and has passed the functional trial.

It complies with the following standards and regulations:

2009/23/CE; 93/42/CE; EN45501

9. NAMEPLATES

 **CAUTION!** The nameplate contains an indication of the manufacturing year ex. 13-2013 and so on.

EN

10. DISCARDING AND DISMISSAL

Stocking

In case of long-time stocking, the parts which could be damaged by dust should be protected.

Dismissal

When the item is no longer being used, it must be made unusable. The dangerous parts must be made safe.

Dismissal EU 2002/96/EC

This product complies with Directive UE 2002/96/CE. The symbol with the crossed bin means that the product, at the end of its life, cannot be treated with domestic waste, but should either be taken to a recycling centre specialised in electric and electronic equipment or returned to the seller while buying an equivalent product. The user is responsible for the correct dismissal of the product. An appropriate recycling, treatment and compatible environmental dismissal helps reducing negative effects on environment and health and promotes the recycling of the materials of which the product is made.

For further details on the available collection methods, contact the local waste dismissal service or the store where the product has been bought.

As a consumer, the user is obliged by law to return used or dead batteries. You can deliver your batteries to the public collection points in your town. It is also possible to give them back to any battery reseller provided with specific containers. Electric and electronic equipment must be delivered to specific collection points also in case of discarding.

NOTE: The following symbols indicate the presence of dangerous substances

Batteries:

Pb Pb = batteries containing lead **Cd Cd** = batteries containing cadmium

Hg Hg = Batteries containing mercury

 **CAUTION! Do not throw electric components and used batteries in the domestic waste**
Deliver the batteries to any local collection centre.

11. WARRANTY

The present certificate should be kept until the warranty expires.

It should be shown together with the invoice, fiscal receipt or delivery note complete with the reseller's name and selling date, every time a technical intervention is required. In case of non-compliance, the user will not be entitled to any warranty right.

The warranty starts from the purchase date and is valid for the period indicated in the catalogue/price list in force.

Warranty means free replacement or repair within the fixed terms of the equipment components which -at the manufacturer's sole discretion- are defective when sold. Wunder will decide whether the item is to be repaired or replaced.

The warranty does not cover:

- damages due to transportation, falls, negligence and tampering
- damages due to inability to use the tool or improper use
- damages due to inadequacy of the electric system or modifications due to environmental, weather or other conditions
- damages due to wrong installation and repairs carried out by non-authorized personnel
- interventions on site for regular checks or supposed defects
- ordinary maintenance and all what can be considered as a normal wear due to usage

The intervention can be refused in case the tool has been changed or transformed in any way

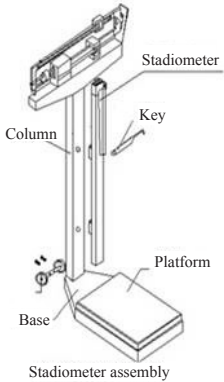
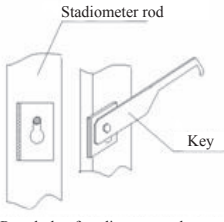
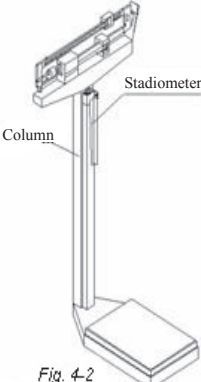
In case of on site intervention, the user should pay the fixed call-out charges. In case the tool is repaired at a Wunder Support Service Centres, the transportation fees and risks will be charged to the user.

Wunder is not responsible for any direct or indirect damage to people, animals or objects caused by a non-compliance with the instructions contained in the present manual or due to improper use.

In case of controversies, the competent jurisdiction is the Court in Bergamo (Italy).


12. ASSEMBLY AND USAGE OF THE MECHANICAL TELESCOPIC STADIOMETER (OPTIONAL)

Refer to FIG.4 to assemble the stadiometer


 <p>Stadiometer Column Key Platform Base Stadiometer assembly Fig 4</p>	 <p>Stadiometer rod Key Rear hole of stadiometer rod Fig 4-1</p>	 <p>Stadiometer Column Fig. 4-2</p>
<p>FIG.4: Insert the stadiometer brackets in the screws on the column</p>	<p>FIG.4-1: Use the provided key to screw the two hexagonal head screws in order to fix the stadiometer to the column</p>	<p>FIG.4-1: the stadiometer is ready for use</p>

Usage of mechanical telescopic stadiometer

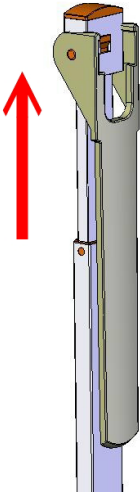
! CAUTION! For height measurement, the patient must go on the scales platform.

MEASUREMENTS HIGHER THAN 131 CM	
	<p>Lift the headrest lever</p>

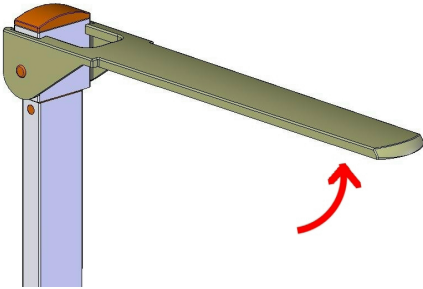Place it in horizontal position

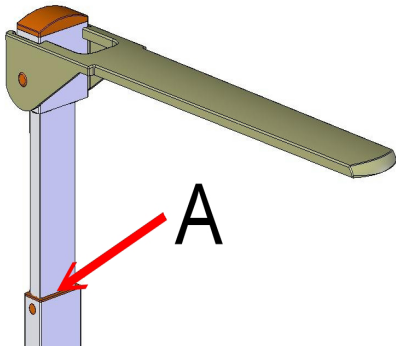
Lower the rod until the headrest lever touches the patient's head.
Height can be red at point 'A'


MEASUREMENTS LOWER THAN 131 CM

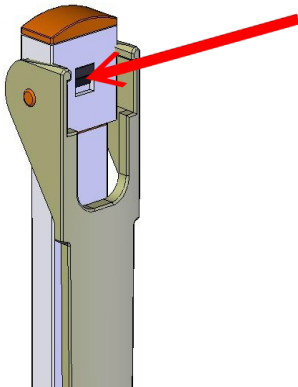
Lower the headrest lever and press the black central locking device

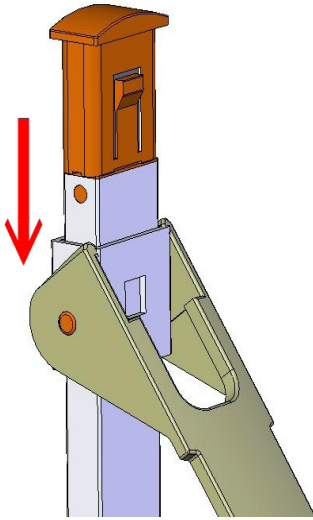Keep the locking device pressed and lower the headrest lever

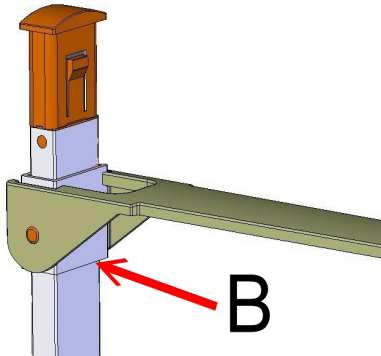Lower the rod until the headrest lever touches the patient's head.
Height can be red at point 'B'

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Wunder Sa.Bi. S.r.l.